mobilux **LED**

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

CE

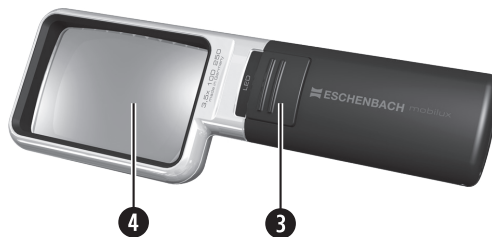
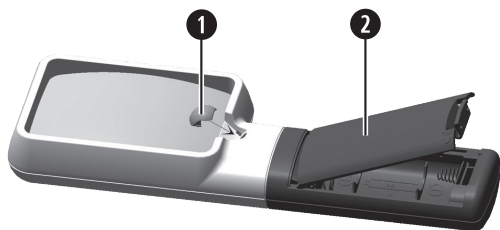
ESCHENBACH

Deutsch	4
English.....	8
Français.....	12
Italiano.....	16
Español.....	20
Nederlands.....	24
Dansk	29
Svenska.....	32
Norsk.....	36
Suomi.....	40
Polski.....	44
Česky	48
日本語.....	52

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English

Introduction

You have acquired a brand name product from Eschenbach that has been manufactured in Germany as per the most modern production processes. Before the first use you should have read through these instructions attentively so as to become familiar with all of the device's functions. mobiluxLED is intended for use as a visual aid, in that the product presents you with a magnified image.

Safety Instructions

- ▶ **This product is not a toy! Do not let children handle it without supervision!**
- ▶ **Risk of asphyxiation! This product contains small parts that could be swallowed!**

- ▶ **Danger of fire! Lenses in optical devices can cause significant damage by the focusing of light to generate heat if handled and stored improperly! Take care that you never leave uncovered optical lenses lying in direct sunlight!**
- ▶ **Danger of glare and injury! NEVER look directly into the sun or other bright light sources with optical devices! Never look directly at the light source on the underside of the housing!**
- ▶ **Protect your illuminated magnifier against impacts of all kinds, moisture and excess heat. Never lay your illuminated magnifier on a heating radiator or in direct sunlight!**
- ▶ **Check the batteries regularly. Leaking batteries can cause damage to the device and it could lead to irritation if the leakage makes contact with the skin.**
- ▶ **Remove the batteries from the battery compartment as soon as they are spent or if the device is to be unused for a long period of time.**
- ▶ **Never subject the mobiluxLED to wetness!**
- ▶ **Ensure that other people, particularly children, are aware of these risks!**

Items supplied

mobiluxLED • Yellow filter • 2 Batteries • Case

Initial device operation

For equable illumination of your reading material this mobiluxLED is equipped with

a high-performance LED, which is powered by 2 batteries (AA/LR6). Before the first use, you must insert the batteries.

1. To do this, open the battery compartment cover (2).
 2. Insert 2 batteries into the battery compartment as per the alignments shown therein. Pay attention to observance of the correct polarities, otherwise the device will not work.
 3. Close the battery compartment.
- Batteries and cells must not be disposed of in household waste! Ensure that they are disposed of properly!

Using the mobiluxLED

You can use mobiluxLED with your reading glasses.

1. Hold the mobiluxLED with the lens centrally over the text that is to be magnified.
2. The optimal imaging quality with mobiluxLED is to be obtained when you maintain the distance to the lens determined for you for the magnification used by you. This value (in mm) is to be found in the chapter “Technische Daten / Technical Data / Caractéristiques techniques” on page 3,  ←  (Spacing lens-eye) as well as above the switch on your mobiluxLED.
3. Switch the LED light on by sliding the shifter (3) in the direction of the lens (4).
4. After use, switch the LED light off by sliding the shifter (3) in the direction away from the lens (4).

Note: If the LED flickers, the batteries are drained and must be exchanged for new ones. In this case, proceed as detailed in the chapter “Initial device operation”.

If the colour of the light is unpleasant for you, you can place the enclosed yellow LED filter cap (❶) over the LED, as shown in the top figure on page 3.

To protect the lens your mobiluxLED is supplied with a case which, by means of a belt loop, can be attached to your belt.

To enable you to hang your mobiluxLED around your neck it has an eyelet located on the end of the handle and through which a lanyard, which is optionally available, can be threaded. (Stock.-No. 15981).

Care Instructions

- ▶ When cleaning the mobiluxLED do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents! The lenses could be destroyed!
- ▶ Do not clean the mobiluxLED in an ultrasonic bath or under flowing water!
- ▶ Clean the mobiluxLED with a soft, lint-free cloth (e.g. with a cleaning cloth for glasses). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults

arising in connection with manufacturing errors or material errors. In the event of damage due to improper handling, including damage from falls or impacts, no warranty claims can be accepted. Warranty claims can only be accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH