Sicherheitshinweise — Pflegehinweise
Safety instructions — Care instructions
Consignes de sécurité — Conseils d'entretien
Avvertenza di sicurezza — Istruzioni per la pulizia
Indicaciones de seguridad — Indicaciones de mantenimiento
Veiligheidsinstructies — Onderhoudsinstructies
Sikkerhedsanvisninger — Vedligeholdelse
Säkerhetsanvisningar — Skötselråd
Sikkerhetsanvisninger — Stell og pleie
Turvallisuusohjeet — Hoito-ohjeet
Wskazówki dotyczące bezpieczeństwa — pielęgnacji
Bezpečnostní pokyny — Pokyny k údržbě
安全なご使用のために — 手入れの方法

 **ESCHENBACH**

Deutsch

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English

Safety instructions

- ▶ Risk of fire!
Lenses used in optical equipment (magnifiers, reading glasses, optical systems, filterglasses and filter clip-ons) may cause considerable damage by their "burning-glass effect" if used or stored improperly. Ensure that optical lenses are never left uncovered in the sunlight!
- ▶ Danger of blinding or injury!
Never look directly at the sun through any optical equipment!
- ▶ Protect optical devices from knocks, strikes, moisture and excessive heat!
Never place optical equipment on a heating element or in direct sunlight!
- ▶ Ensure that other people, especially children, are aware of these dangers!

Care instructions

- ▶ Do not use any soap solutions that contain softeners, alcohol-based or organic solvents or abrasive cleaning products! These may damage the lenses.
- ▶ Do not clean optical equipment in an ultrasonic bath or under running water!

Clean your optical equipment with a soft, lint-free cloth (e.g. spectacle cleaning cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. fingerprints).

Filterglasses and filter clip-ons for protecting the eyes from sunlight comply with the European Standard EN 1836:2005+A1:2007 (D).

Filtercategory for filter glasses and filter clip-ons

Product (part.-no.)

filter category

Cut-off filter glasses:

(16603801, 16603802)

3

(16604501, 16604502)

1

(16605111, 16605112)

3

(16605271, 16605272)

2

Cut-off filter clip-ons:

450nm (1662450)

0

511nm (1662511)

1

527nm (1662527)

2

550nm (1662550)

2

450nm, polarized (1662450P)

2

511nm, polarized (1662511P)

3

527nm, polarized (1662527P)

3

550nm, polarized (1662550P)

3

8

Product (part.-no.)	filter category
wellnessPROTECT:	
filter 15%	1
filter 65%	2
filter 50-15%	2
filter 85%	3

When glazing with glasses for individual refraction the wellnessPROTECT frame must not be heated!

polarized clip-ons:

brown (29211, 29261)	3
grey (29212, 29262)	3
green (29213, 29263)	3

= not suitable for driving motor vehicles

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Eschenbach Optik GmbH